

LIGA SKIN™ Skin Stapler



35 Clips

Easy operation

Shorten suturing time

POSITIONING

The **LIGASKIN™** Skin Stapler is designed to be fired from multiple angles. It has an ejector spring that automatically disengages the staple to facilitate instrument removal in any direction.

STAPLE FORMATION

The flat, dimpled staple top span is designed to ensure proper staple formation. To use the skin stapler properly squeeze the trigger completely and release. Do not pull or lift up the stapler when releasing the trigger. Failure to completely squeeze the handle can result in incomplete staple formation and insufficient wound approximation.

ORDER INFORMATION

10 units / Box

AXON

Medical Solutions

India Corporate Office

AXON Medical Solutions Pvt.Ltd.

📍 Office No: 29, 1st Floor,
Sushant Vyapar Kendra, Sushant Lok-I,
Gurugram, PIN - 122002, Haryana [INDIA]

☎ : +91 95990 17699 / 93111 45832 / 73308 13300

☎ : +91 124 4257540

✉ : info@axonmedicalsolutions.com

🌐 : www.axonmedicalsolutions.com

